\*All pictures & color are for reference only.  
Please refer to the actual product received.

## **Y-105/Y-105W Household Oxygen Concentrator**

### **User Manual**

Please read and understand this manual carefully before using the product.

## **Warning:**

**In the event of a power failure or oxygen concentrator malfunction, persons in urgent need of oxygen and critically ill patients shall be provided with alternative oxygen source.**

### **I. Before Installation**

- Keep the oxygen concentrator upright! The angle of inclination should not exceed 5°.
- If the power supply voltage is unstable, please use a voltage regulator (2000W)! (Unstable voltage usually occurs in the countryside.)
- Please select a safe and qualified socket and terminal block (1100W is better)!
- Non-professionals are not allowed to open the case!

### **II. Placement**

- Place the oxygen concentrator in a well-ventilated area out of direct sunlight, and keep it at least 10 cm away from walls and other objects.
- Do not place the oxygen concentrator near heat sources, open flames or potential fire; in wet or unsheltered areas; or in too high ( $\geq 55^{\circ}\text{C}$ ) or too low ( $\leq -20^{\circ}\text{C}$ ) temperatures.
- Never put any heavy objects, sundries or containers with water or oil on the oxygen concentrator.
- Never put sundries on the bottom of the oxygen concentrator to avoid blocking the inlet and outlet, or the high temperature may cause shutdown or low oxygen concentration.

### **III. During Use**

- Oxygen is flammable, so avoid smoking near the operating oxygen concentrator and keep it away from open flames and potential fire hazards to prevent fire incidents.
- Do not restart the oxygen concentrator frequently. Restart it at least 30 seconds after shutdown if necessary.
- Do not apply oil or grease to any part of the oxygen concentrator to avoid pollution or fire accidents.

# Product Characteristics

---

## I. Principle of Oxygen Supply

The oxygen concentrator adopts international advanced Pressure Swing Adsorption (PSA). Using air as the raw material, it operates at room temperature and absorbs nitrogen through molecular sieve of different molecular diameters, allowing oxygen of 90% purity to be separated from the air for use.

## II. Structure Features

- All Plastic Shell: Safe and reliable.
- Time Cumulative Function: Showing the total running time on the display.
- Timer: Shutting down when the set time is reached.
- Remote Control Function: Being controlled remotely for convenience.
- One-Button Call for Help Function: The remote control is equipped with a "Call" button. After the button is pressed, the machine will emit the "I need help" sound.

# Product Characteristics

---

## III. Main Technical Specifications

1. Power Supply:  AC220V±10% / AC230V±10% 50Hz  
 AC110V±10% 60Hz or 50Hz (in Japan)
2. Rated Power: 150W
3. Oxygen Flow: 1, 2, 3, 4, 5, 6, 7 L/min
4. When the oxygen flow is 1L/min, the oxygen purity is 93%±3% (V/V) (reach the specified purity level within 15 minutes of the initial start).  
Other than that,  
Oxygen purity: 35±3%~78±3% (V/V) (Oxygen flow: 2,3,4,5,6,7 L/min)
5. Noise: ≤45dB (A)

Measured at 1m from the geometric center point of the front of the equipment under normal start-up conditions (without adding water to the humidification bottle).

6. Net Weight: ≈6kg
7. Size: ≈30.5cm×18cm×31cm/12"\*7.1"\*12.2"
8. Safety System:
  - Current Overload Protection
  - Compressor Shutdown at High Temperature
9. Electrical Classification: II
10. Operating Mode: Continuous
11. Normal Working Environment
  - Ambient Temperature: 5°C~40°C;
  - Relative Humidity: ≤80%;
  - Atmosphere Pressure: 86kPa~106kPa

**Note: If the storage temperature is lower than 5°C, the equipment shall be placed at normal operating temperature for 4 hours before using.**

12. Production Date: See the product nameplate

Please avoid violent collision and overturning during transportation.

The machine should be stored in a well-ventilated room without strong sunlight and corrosive gas.

# Installation and Use

## I. Inspection After Unpacking

Before installation and operation, check whether the appearance of the equipment is in good condition, and whether the quantity of accessories is as shown in the Packing List in this manual. If anything is missing, please contact us.

## II. Outline Diagram and Function Introduction

Figure 1: Front side view of the machine

Figure 2: Rear side view of the machine

# Installation and Use

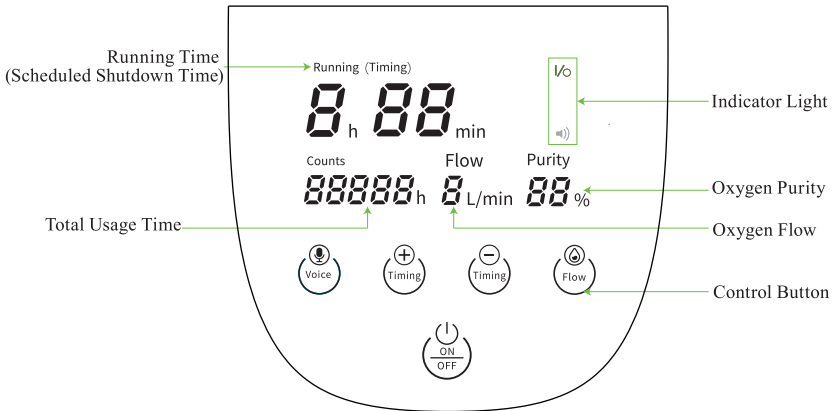
Figure 3: Schematic diagram of the display & control panel

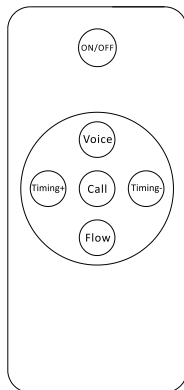
Figure 4: Remote control

## III. Preparation

- Gently pull out the humidification bottle in the direction shown in Figure 5, remove the rubber cap at the top, and fill the bottle with cold purified water not exceeding the maximum water level.

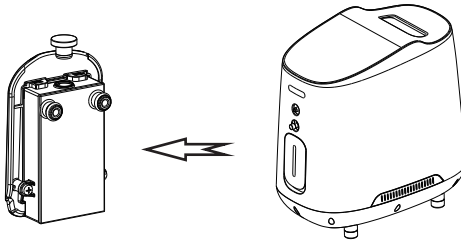
Figure 5

Note: The machine can provide a regular oxygen concentration without water in the humidification bottle.

- Close the humidification bottle with the rubber cap, and install it in the machine.
- Connect the oxygen concentrator to the power supply: Connect one end of the power cord to the machine's power supply port, and the other end to a power outlet.
- Press the power switch to "I" to enter standby state. Then press the "ON/OFF" button to start normal operation. When the oxygen concentrator is operating, it will make a "Chi~" sound, which is a normal breathing sound, every few seconds.

## IV. Oxygen Inhalation Operation

- Press "Flow" on the control panel to adjust the oxygen output flow as required. Ensure that there is oxygen flowing from the outlet.
- Connect the nasal cannula to the oxygen outlet. Then adjust the nasal cannula to a proper position for inhalation.

## V. Timing Function Operation

**The oxygen concentrator has a timing function, allowing the automatic shutdown time to be set within 8 hours.**

- On initial startup, the display shows "Running", indicating that the timed shutdown function is not set and the machine is in continuous operation until manually switched off.
- Press the "Timing +" button to change "Running" to "Timing". Press the "Timing +" button again to increase the shutdown time by 10 minutes with each press, while pressing the "Timing -" button reduces the time by 10 minutes. The set time will be locked in once you stop pressing the buttons, and it will count down as the machine runs.
- When the set time is reached, the oxygen concentrator will automatically shut down, with the display showing "Timing --h--min". Press the "ON" button to restart.

## VI. Voice Operation

- Press the "Voice" button on the control panel to turn on or off the voice.

## VII. Shut Down

- If it is necessary to stop oxygen inhalation temporarily during use, press the "ON/OFF" button on the control panel to stop the oxygen supply. Press the button again to start.
- After use, press the "ON/OFF" button to stop the oxygen supply, turn off the power switch, and disconnect the power supply.

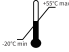
# Installation and Use

## VIII. Wireless Remote Control

When the oxygen concentrator is in operation or standby mode, users can start, adjust, or shut it down using the wireless remote control.

**Note: The remote control should be in front of the machine.**

## IX. Symbols and Significations of Safety Requirements in This Machine

Symbols	Significations	Symbols	Significations
	Alternating current		Consult the manual
	ON (power)		OFF (power)
	Keep this side up		Fragile
	Keep dry		Class II equipment
	Stacking height		Temperature limit

## X. How to Use the Oxygen Inhaler

### ➤ Introduction to the Oxygen Inhaler

Nasal cannula: Can be hung on the ears, easy to operate and use.

### ➤ Method of Use

1. Before oxygen inhalation, connect the nasal cannula to the oxygen outlet. Then adjust the nasal cannula to an appropriate angle and wear it as shown in Figure 6.

## Installation and Use

---

2. Wearing method of the nasal cannula (see Figures 6 and 7)

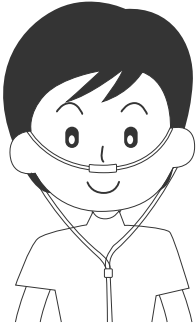
Figure 6

Figure 7

3. Cleaning the nasal cannula: Clean once a week using a mild detergent, then rinse with clean water. Ensure the hoses are dry before use.

4. Cleaning the nasal cannula's oxygen inhalation heads: Wipe with medical alcohol.

### **XI. How to Use the Atomizer (Y-105W)**

Add the liquid medicine into the atomizing cup, and unscrew the atomizing cap of the atomizer on the machine counterclockwise for atomizing. Connect the atomizer and atomizing cup with an air catheter. When the atomizing cup begins to produce mist, hold the atomizer mouthpiece on the cup in your mouth and start the atomizing treatment by mouth breathing.

# Maintenance and Service

**Warning: Disconnect the power supply before maintenance of the oxygen concentrator. Do not disassemble the housing to avoid electric shock.**

## I. Clean the Housing

Keep the housing clean: Disconnect the power first, then use a damp and soft cloth soaked in sterilizing fluid to clean. Prevent any fluid from entering the gaps of the housing.

## II. Replace Filter

The replacement of the filter is very important to protect the compressor and molecular sieve and to prolong the service life of the oxygen concentrator. Please replace the filter after 300 hours of use.

**Warning: Do not operate the oxygen concentrator without the filter installed in place, which will cause long-term damage to the machine.**

- Pull out the filter cover (see Figure 8);
- Pull out the filter, and replace it with a new one (see Figure 9);
- Put the filter cover back in place.

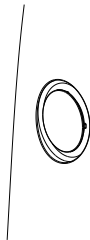
Figure 8

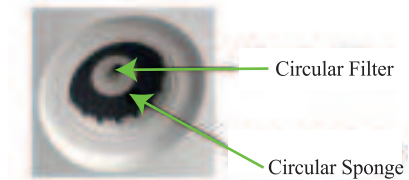
Figure9

## Maintenance and Service

The actual replacement interval of the filter shall be determined by the actual usage time and the environment. If the filter becomes black or dirty, it shall be replaced immediately, regardless of the usage time.

### III. Clean the Humidification Bottle

- Replace the water in the humidification bottle every day.
- Clean the humidification bottle once a week, and rinse it with clean water repeatedly to ensure oxygen sanitation. Be sure to clean the rubber cap when cleaning the bottle.
- Disassemble the humidification bottle for thorough cleaning.

When cleaning, pay attention to the core tube and the aerator at the bottom to ensure a smooth flow of oxygen.

Figure 10

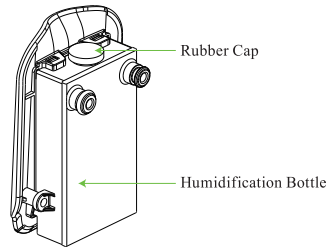
Figure 11

### IV. Replace Fuse

Before operating, ensure the power is disconnected. The fuse holder is located at the bottom of the machine. Use a cross-head screwdriver to turn counter-clockwise and remove the fuse holder cover, and then replace the fuse.

(Note: Fuse Model: 220V~: T1.6AL 250V/110V~: T2.5AL 250V,  $\phi 5 \times 20$ )


## Maintenance and Service

### V. Common Troubleshooting

Problems	Possible Causes	Solutions
The oxygen concentrator is connected to the power supply, but it does not work and the display shows nothing	1) Broken fuse in the machine	1) Replace the fuse tube
	2) Insufficient power at the power outlet	2) Do not use extension cords to move the oxygen concentrator to another power outlet or circuit
	3) If the problem is not solved, contact us	
The oxygen concentrator works, but there is little or no air output at the oxygen inlet	1) Kinked or blocked nasal cannula hose	1) Straighten the nasal cannula hose
	2) Loose or missing rubber cap on the humidification bottle	2) Tighten the rubber cap
	3) Blocked humidification bottle	3) Clean the humidification bottle
	4) Dirty filter	4) Replace the filter
	5) If the problem is not solved, contact us	
The oxygen concentrator is connected to the power supply, but it does not work and the display shows “fault”	1) Compressor shutdown for overheating protection	1) Shut down and restart the machine after it cools down
	2) If the problem is not solved, stop using the machine and contact us immediately	

**Note: If the oxygen concentrator malfunction cannot be resolved, please contact us.**

### I. Air Line Principle:

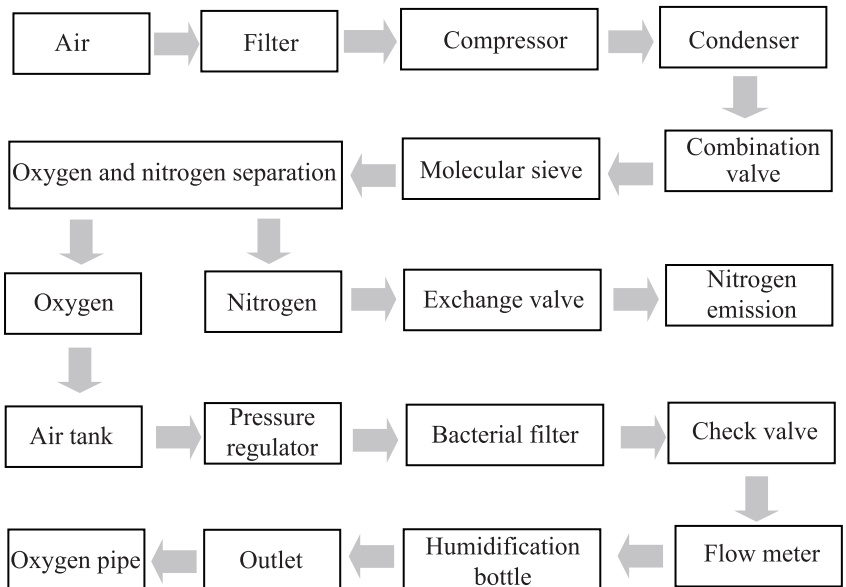### II. Environment of Use

In order to prevent the machine from being interfered by the strong electromagnetic environment or other electrical products, please keep the machine at least 1 meter away from other household appliances in use.

**III. Non-professionals are not allowed to disassemble the housing of the oxygen concentrator. If professional maintenance personnel require the electrical diagram of the product, please contact our after-sales service.**

### IV. After-Sales Service

- Within six months of purchase, if the oxygen concentrator is used and kept under normal conditions, it can be repaired or replaced free of charge when it has any quality problem.
- Within a year, if it has any malfunction, paid-for parts and free repairs will be provided.
- The following are beyond the warranty: (1) Damage caused by misuse or use in abnormal operating environments; (2) Damages caused by non-compliance with the user manual and improper storage; (3) Damages caused by arbitrary disassembly, maintenance, modifications, or other unauthorized repairs; (4) Damages caused by improper transportation after sale; (5) Losses caused by force majeure (flood, fire, lightning strike and earthquake).
- Circuit diagram and documents for maintenance can be supplied if required. For any inquiries about electric circuit troubleshooting, please contact us.

### V. Packing List

Number	Part Name	Quantity
1	Household Oxygen Concentrator	1
2	Power Cord	1
3	Nasal Cannula	1
4	Remote Control	1
5	Fuse	2
6	Atomizer Kit (Y-105W)	1
7	Circular Sponge	2
8	Circular Filter	2
9	User Manual	1
10	O-ring	4

### VI. Disposal of Waste and Residues

Dispose of waste and residues in accordance with relevant national laws and regulations.

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