

VEVOR

Affordable. Reliable. Home Improvement.

Tool Sanitizer

Technical Support and E-Warranty Certificate
www.vevor.com/support

VEVOR




Affordable. Reliable. Home Improvement.

Tool Sanitizer

Model: 208B



This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Symbol	Symbol Description
	<p>Warning: To reduce the risk of injury, the user must read the instructions manual carefully.</p>
	<p>This symbol, placed before a safety comment, indicates a kind of precaution, warning, or danger. Ignoring this warning may lead to an accident. To reduce the risk of injury, fire, or electrocution, please always follow the recommendation shown below.</p>
	<p>CORRECT DISPOSAL:</p> <p>This product is subject to the provision of European Directive 2012/ 19/EC. The symbol showing a wheelie bin crossed through indicates that the product requires separate refuse collection in the European Union. This applies to the product and all accessories marked with this symbol. Products marked as such may not be discarded with normal domestic waste, but must be taken to a collection point for recycling electrical and electronic devices.</p>

FCC INFORMATION

CAUTION: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment!

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- 1) This product may cause harmful interference.
- 2) This product must accept any interference received, including interference that may cause undesired operation.

WARNING: Changes or modifications to this product not expressly approved by the party responsible for compliance could void the user's authority to operate the product.

SAFETY INSTRUCTIONS




WARNING: Do not expose eyes or skin to UV light.

1. When using the sterilizer for the first time, operate it in an empty load mode for working cycle, then open the door for 20 minutes. This operation can eliminate any odd smell inside the cabinet.
2. Make sure to drain all water the utensils before placing them in the sterilizer. The utensils should not be stacked but placed vertically one by one to achieve the best sterilization effect.
3. In case of any abnormal conditions (such as unusual noise, odd smell, or smoke), immediately unplug the power cord of the sterilizer and contact the repair center or the manufacturer for inspection and repair. Do not use it again until the cause of the fault is resolved.
4. When maintaining or cleaning the device, make sure to unplug the power cord to ensure complete disconnection from the power source.
5. Wait for 10 minutes after the sterilization cycle completed before opening the door. Use the utensils only after they have cooled down to avoid affecting the sterilization effect or causing burns.
6. Do not install sterilizer near flammable materials.
7. If the power cord is damaged, it must be replaced by a professional from the manufacturer, its repair department, or similar department to avoid danger.
8. Do not place the sterilizer in areas prone to moisture or water splashes, and do not use heating appliances near the sterilizer, as this may cause deformation or malfunction of the sterilizer.
9. Special user groups (such as children) are prohibited from using this device.
10. After use, if it will not be used for a long, please cut off the power and do not leave the power cord in a connected state to avoid danger.
11. It is strictly prohibited to directly rinse the with water or immerse the machine in water.
12. The smell produced during the ultraviolet disinfection process is the smell of active oxygen, which is normal phenomenon and harmless to the human body.

MODEL AND PARAMETERS

Model	208B
Input	AC 110V 60Hz(US), AC 220-240V 50Hz(EU&AU)
Max. Power	12W
Time setting	5-60 min
Size	397*272*305mm

STRUCTURE DIAGRAM

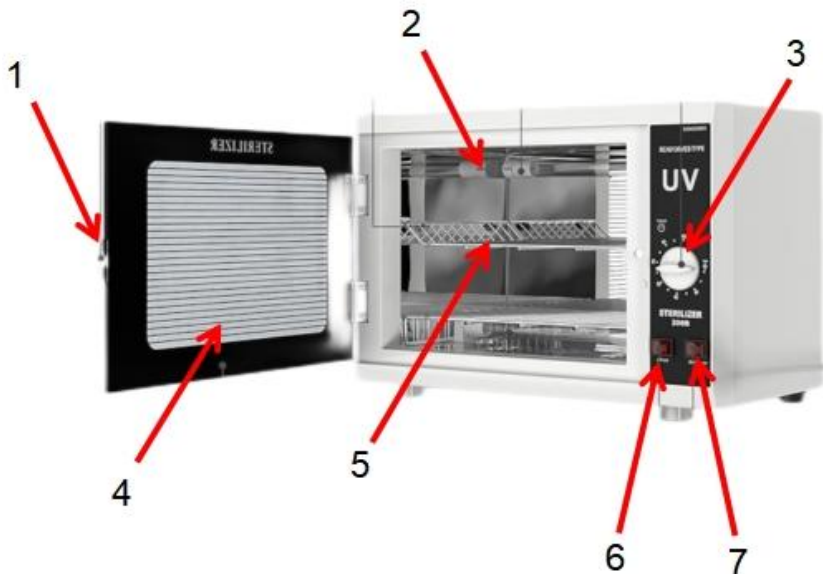

1.Handle	2.Uv lamp
3.Time adjustment knob	4.Tempered glass door
5.Steel basket	6.Upper light button
7.Lower light button	

PARTS LIST

	
Tool Sanitizer x1	Steel basket x2
	
Power Cord x1	User Manual x1

OPERATION METHOD

- (1) Clean the articles to be disinfected in advance and then put them into the disinfection cabinet. Close the cabinet door; The door must be in contact with the micro-switch for the lamp tube to work normally.
- (2) Insert the power supply and twist the time button to select the required disinfection time. The time setting range is from 5 to 60 minutes.
- (3) After selecting the corresponding disinfection time, open the upper light button and lower light button, the indicator light lights up to indicate that the machine enters work State, the disinfection lamp tube in the disinfection cabinet lights up.
- (4) During the work of the disinfection cabinet, it is necessary to take articles and only need to open the cabinet door. The disinfection lamp in the disinfection cabinet goes out automatically, and the cabinet door lights up automatically when closed. The poison time is automatically extinguished when it is completed.

Maintenance

1. Make sure to unplug the power cord before cleaning or maintaining the disinfection cabinet.
2. Wear gloves cleaning or maintenance to prevent cuts from sharp edges inside the cabinet.
3. Regularly wipe off stains from the surface to keep the cabinet clean and attractive.
4. Electrical components must not be immersed in water. Do not rinse the cabinet with water.
5. Regular maintenance is recommended. Unplug the power cord, use a cloth with a small amount of neutral detergent to gently remove stains inside and outside the cabinet, then use a clean damp cloth to remove the detergent.
6. When disinfection cabinet is not in use, it should be unplugged. If it is to be unused for a long time, moved, or serviced, make sure to unplug the power cord first.
7. If any abnormality occurs during use, stop using it immediately and unplug the power cord. To avoid danger, installation or repair must be done by a qualified electrical repair person.

Common Fault Guidelines

Fault Description	Cause Analysis	Solution
The device is not powered on	The power cord is not plugged in	Plug the matching power source we provide
The device is not functioning	The time adjustment knob is not adjusted	Adjust the time adjustment knob

Manufacturer: Foshan Kangzhiqing Electrical Appliance Co.,Ltd

Address: No.23-2,Liuyi San Road,Nanhai District,Foshan City,Guangdong Province, China

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

UK	REP
-----------	------------

YH CONSULTING LIMITED. C/O YH Consulting Limited Office 147, Centurion House, London Road, Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
-----------	------------

E-CrossStu GmbH
Mainzer Landstr.69,
60329 Frankfurt am Main.



Technical Support and E-Warranty Certificate
www.vevor.com/support